



Quality care,  
from anywhere

Introducing Cochlear™  
Remote Check








Remote Check is a convenient, at-home testing tool that allows you to complete a hearing review using your a compatible smartphone\*—without having to travel to the clinic.

It's never been easier or more convenient for your clinician to check your hearing health, and ensure that you continue to hear your best.

All you need is the Cochlear™ Nucleus® 7 or Kanso® 2 Sound Processor and the Nucleus Smart App.\* Remote Check is a feature within the Nucleus Smart App.

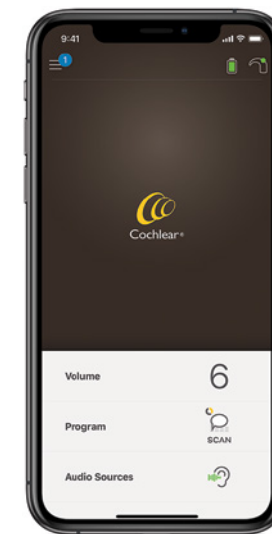


Nucleus Smart App



Kanso 2 Sound Processor




Nucleus 7 Sound Processor

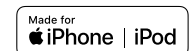



\*The Remote Check feature of the Nucleus Smart App is available on iPhone and iPod touch. For compatibility information visit [www.Cochlear.com/Compatibility](http://www.Cochlear.com/Compatibility).





## How Remote Check works


 Download the Nucleus Smart App on your compatible smartphone.\* You will also need to pair your sound processor(s) to your device, before you can use the app's Remote Check feature.

 Your clinician will activate the Remote Check feature of the app so you can access it. You will receive a notification of your enrollment via email and through a notification in the Nucleus Smart App.

 Log in to the Smart App at your convenience to complete the activities your clinician has recommended for you. The full suite of Remote Check activities for adult patients are designed to be completed in as little as 15 minutes,\*\* although it may take longer for some patients and children. Remote Check is designed so that it can be completed at your own pace.

 Once you've completed the activities and answered a few questions, your results will be sent directly to your clinician via a secure connection.

 Based on your results and the information you provided your clinician will determine the next steps and will reach out to you directly to discuss in more detail.

 If you do need a face-to-face appointment, you can contact your clinician to ensure you receive the care you need.

\* The Remote Check feature of the Nucleus Smart App is available on compatible smartphones. For compatibility information visit [www.cochlear.com/compatibility](http://www.cochlear.com/compatibility).







\*\* D1739391- Remote Check Usage Investigation. Data on file. The median time taken is 20 mins for a unilateral patient and 30 mins for a bilateral patient.



## What you need to do

Simply select the Remote Check feature on your Nucleus Smart App, and complete the activities you have been assigned by your clinician. These activities are similar to those you'd go through with your clinician if you were at the clinic, and can take as little as 15 minutes\*1 to complete.



- 1 Implant site photos**  
Take photos of the area where you wear your coil and sound processor. 
- 2 Questionnaire: part 1**  
Complete a questionnaire to provide an indication of your hearing health and needs. 
- 3 Audiogram**  
Take a simple hearing test to indicate which soft sounds you can and cannot hear. 
- 4 Speech in noise**  
Take a speech-in-noise test to measure how well you can understand speech when there is noise in the background. 
- 5 Questionnaire: part 2**  
Complete the second part of the questionnaire, which relates to listening activities and training. 
- 6 Impedance Check**  
The app will also check how your implant is functioning, and send the results (along with other relevant device data) to your clinician. 

\*The median time taken to complete all five activities is 20 minutes for a unilateral recipient and 30 minutes for a bilateral recipient.1

1. D1739391. Remote Check Usage Investigation. Clinical Investigation Report (CIR). Data on file. April, 2020.





## **The future of hearing health starts here**

If you have a Nucleus 7 or Kanso® 2 Sound Processor and want to have a convenient, remote hearing review that can happen at a time or place that works for you, ask your clinician about Remote Check today.

Remote Check needs to be enabled by your hearing professional. Contact Cochlear or your hearing health professional to learn more.



# Hear now. And always

As the global leader in implantable hearing solutions, Cochlear is dedicated to helping people with moderate to profound hearing loss experience a life full of hearing. We have provided more than 600,000 implantable devices, helping people of all ages to hear and connect with life's opportunities.

We aim to give people the best lifelong hearing experience and access to innovative future technologies. We collaborate with leading clinical, research and support networks.

That's why more people choose Cochlear than any other hearing implant company.

Please seek advice from your health professional about treatments for hearing loss. Outcomes may vary, and your health professional will advise you about the factors which could affect your outcome. Always read the instructions for use. Not all products are available in all countries. Please contact your local Cochlear representative for product information.

Remote Check is intended for ages 6 and older. The Remote Check feature is only visible and accessible if the feature is enabled by a clinician. Clinicians should consider the suitability of the feature before enabling Remote Check.

Remote Check does not replace clinical care and does not involve remote programming of the sound processor.

Cochlear Nucleus Sound Processors are compatible with Apple and Android devices. The Cochlear Nucleus Smart App is available on App Store and Google Play. For compatibility information visit [www.cochlear.com/compatibility](http://www.cochlear.com/compatibility).

©2020. Apple, the Apple logo, FaceTime, Made for iPad logo, Made for iPhone logo, Made for iPod logo, iPhone, iPad Pro, iPad Air, iPad mini, iPad and iPod touch are trademarks of Apple Inc., registered in the U.S. and other countries. App Store is a service mark of Apple Inc., registered in the U.S. and other countries.

©Cochlear Limited 2020. All rights reserved. Hear now. And always and other trademarks and registered trademarks are the property of Cochlear Limited or Cochlear Bone Anchored Solutions AB. The names of actual companies and products mentioned herein may be the trademarks of their respective owners.

**Cochlear Americas**  
10350 Park Meadows Drive  
Lone Tree, CO 80124 USA  
Telephone: 303 790 9010  
Support: 800 483 3123

**Cochlear Canada Inc.**  
2500-120 Adelaide Street West  
Toronto, ON M5H 1T1 Canada  
Support: 800 483 3123

[www.cochlear.com/us](http://www.cochlear.com/us)